


La science pour la santé \_\_\_\_\_ From science to health

## COVID19 - FFP2 and FFP3 masks

## (personal protective equipment at the laboratory)

Your employer, the director of your research unit or your research structure is in charge of it	
Who supplies it?	Available at your workstation
	This mask is a personal protective equipment (PPE) FFP2 or FFP3
	NF EN 149 standard
What kind of mask?	Standard Filtration class CE marking Mask FFP2 INRS ED6106
Where to wear it?	At your workstation, following the occupational risk assessment, if your tasks may expose you to some hazard, especially a biological one
Why to wear it?	<u>Protect yourself</u> from an external contamination you would be exposed to in your professional activities
How to wear it?	<ul> <li>Read the following figure and remember these basic instructions:</li> <li>→ Wash your hands with water and soap or use a hydro-alcoholic solution before putting on and after removing the mask</li> </ul>
	Hold the mask by its strings to put it on
	Be sure the mask is fitted and covers mouth and noze
	For removal : release strings to remove the mask from your face. Do not touch the interior or the exterior of it



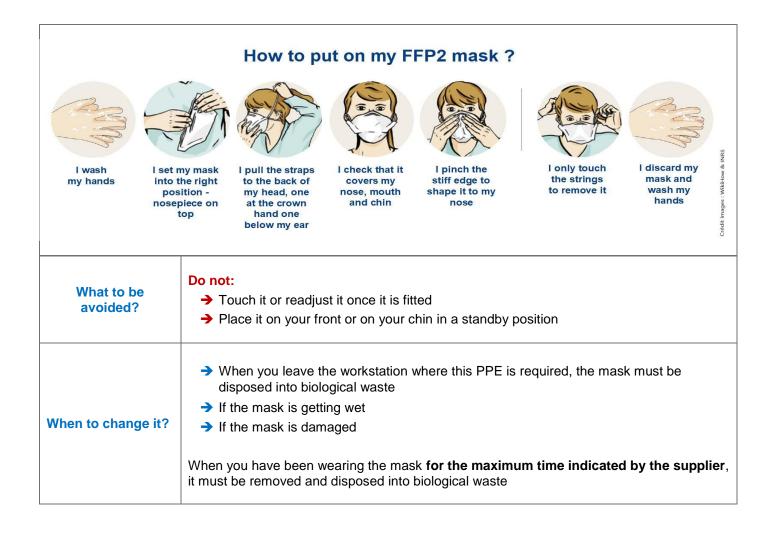

## Warning : this mask is no substitute for the barrier gestures.

It only adds a physical barrier, when you are in close contact with other people



Stay at a distance of 1 m at least (3 feet) – or in a safe area of 4 m<sup>2</sup> (45 sqf)



Wash your hands regularly with water and soap, or use a hydro-alcoholic solution,

particularly when arriving at work, before lunch, after getting to the batheroom, after sneezing / coughing / blowing your noze, anytime you manipulate your face mask



Cough or sneeze inside your elbow When you are face-to-face with a coughing person, step back and stay at a range of 1 m at least (3 feet)



Use a disposable handkerchief and throw it away



Do not touch your face



Greet one another without shaking hands or kissing each others